## TUCKY KASUTUSJUHEND

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## Tehnilised omadused

Type of product	Direct mode clinical thermometer		
Thermometer references Adhesives reference	- TUCKY-21, TUCKY-21W, TUCKY-21Y, TUCKY-21O, TUCKY-21P, TUCKY-21B & TUCKY-21N - ADH-21-E1		
Storage / transport conditions	For complete product (including adhesives) or for adhesives only: Temperature: +10 to 27 °C (+50 to 80 °F); Relative humidity: 40 to 60 %.		
	For thermometer only: Temperature: -20 to +55 °C (-4 to 131 °F); Relative humidity: 15 to 95 %		
Normal conditions of use	Temperature: +15 to 40 °C (59 to 104 °F); Relative humidity: 15 to 95 % (non-condensing) Atmospheric pressure: 76 to 106 kPa		
Display resolution	0,1 °C (0.2 °F)		
Displayed temperature range	+24 to 44 °C (75 to 111 °F)		
Accuracy (maximum laboratory error within the display temperature range)	±0.1 °C (0.2 °F) from 37 to 39 °C (98 to 102 °F); ±0.2 °C (0.3 °F) from 32 to 37 °C (90 to 98 °F) and from 39 to 42 °C (102 to 107 °F); ±0.3 °C (0.5 °F) beyond these ranges.		
Power supply	Rechargeable internal Lithium battery not accessible. Capacity: 35 mAh		
Charger cable	USB 2.0 Type A male to Micro USB Type B male cable		
Charging voltage	5V		
Alerts and information	High temperature: >38.4 °C (101.1 °F) (modifiable), Rollover on the belly (deactivated), Battery discharged (<6 %), Tucky detached, Long connection loss (>15min), Error		
Bluetooth® technology	BLE 4.2; very low output power (1.35 mW)		
Protection class IP22	Protected against solid bodies larger than 12 mm Protected against dripping water up to 15° from vertical		
Supplied accessories	Instruction for use, charger cable, double-sided adhesive tape ADH-21		
Lifespan	Thermometer: 5 years (500 charge cycles); Adhesives: 5 years		
Required equipment	Smartphone or tablet with Bluetooth and internet connectivity. A list of recommended equipment is available on the manufacturer's website		
Compatibility	From iPhone 4S, iPad 3 and Android 4.3		
The device is intended for use in the electromagnetic environment defined below			
ELECTROMAGNETIC EMISSION TESTS		COMPLIANCE	ENVIRONMENT
RF emission (CISPR 11)		Group 1	RF emissions are very low and are not likely to cause interference to nearby electronic equipment
		Class B	Conform
Harmonic current emission		Not applicable	Battery powered
Voltage fluctuation, Emission of Flicker flicker		Not applicable	Battery powered
IMMUNITY TESTING		TEST LEVEL ACCORDING TO IEC 60601	CONFORMANCE LEVEL
Electrostatic discharge (ESD) IEC 61000-4-2		± 8 kV on contact ± 2,4,8 and 15 kV in air	Compliant
Radiated RF IEC 60601-4-3		10 V/m from 80 MHz to 2,5 GHz	Compliant
Magnetic field at mains frequency IEC 60601-4-8		30 A/m	Compliant

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Tootja: e-TakesCare SAS, 35 rue des Chantiers, 78000 Versailles, France